

ATTO ACCESSORIES are created specifically to enhance, support and work with the ATTO mobility scooter in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

ATTO MORE FOR

movinglife®




ATTO
BACKPACK

All you need every day!

A


ATTO BACKPACK STORAGE FEATURES

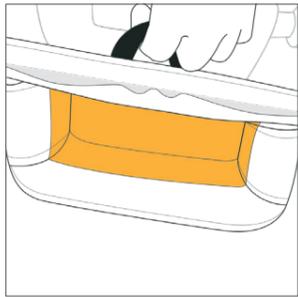

1 Spacious interior with dedicated laptop/tablet compartments, pockets and organizers




2 Dedicated ATTO battery compartment with separate side access (divider folds away when not in use)

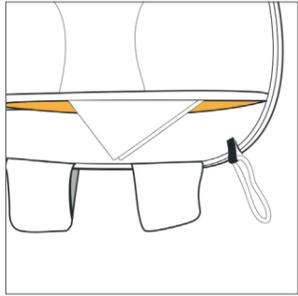


3 Secure pocket for valuables

B


CARRYING ATTO BACKPACK IN FRONT UTILITY BASKET MODE

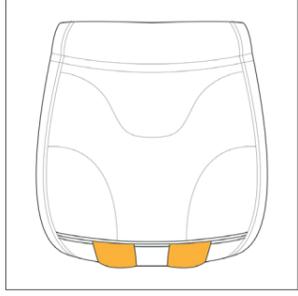

1 Harness straps are safely tucked away behind back padding when not in backpack carrying mode

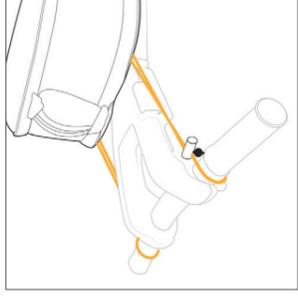

2 Connect to the front handlebar by simply threading the loops over the hand grips and then tighten cable buckles to secure position

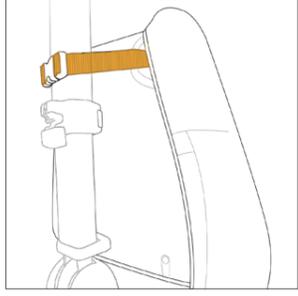

3 Secure lower connecting strap to tiller and buckle it tight, tuck excess strap under backpack rear mesh cover




4 Tuck opening flaps out of the way to enjoy on-hand access and storage

C

CARRYING IN REAR STORAGE & TRANSPORTATION MODE



1 Harness straps are safely tucked away behind back padding when not in backpack carrying mode

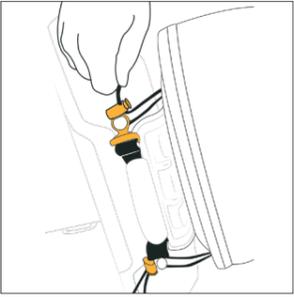

2 Connect the 2 loops to the hooks on the backrest and tighten cable buckle to secure Backpack in best position/release to extend cable loop length and lower backpack position to enable easy handle extraction when ATTO is folded



3 Secure the seat connecting strap and buckle it tight




4 Enjoy safe storage and transportation

D

CARRYING ATTO BACKPACK IN BACKPACK MODE



1 Untuck harness straps and connect to lower buckles

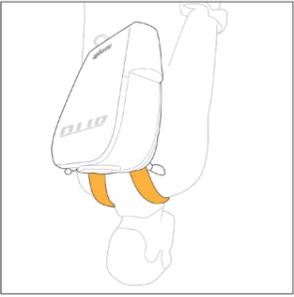

2 Enjoy well padded back and harness comfort