



ATTO ACCESSORIES are created specifically to enhance, support and work with the ATTO mobility scooter in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

**MORE FOR ATTO**

movinglife®




**ATTO**  
ESSENTIALS POUCH

Essentially all you need!

# A

## ATTO ESSENTIALS POUCH IN ATTO CONNECTED MODE

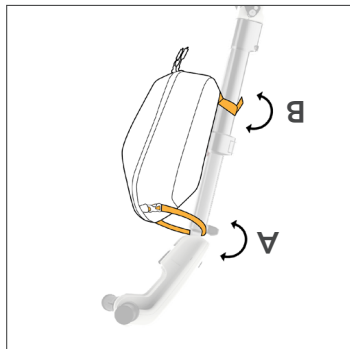

1

**A.** Wrap the top strap around the small gap under the driving console and secure the buckle and secure the lower Velcro strap around the tiller



2

Once attached you can position ATTO Essentials Pouch in any direction and adjust tiller height freely




3

Position pouch facing outward to stay connected when ATTO is folded


# B

## ATTO ESSENTIALS POUCH IN SLING CARRYING MODE

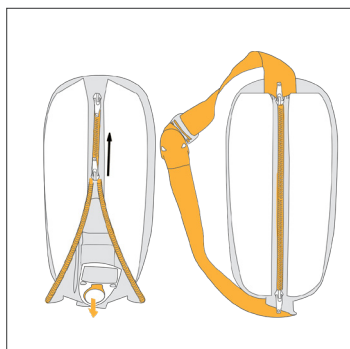

1

Extract sling strap from behind zippered mesh cover and zip back up (note that you will need to unbuckle and pull out one end from under the mesh cover)

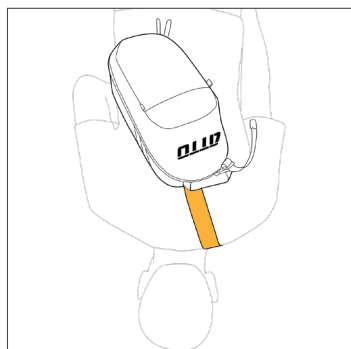

2

Enjoy comfortable sling carrying