

ATTO ACCESSORIES are created specifically to enhance, support and work with the ATTO mobility scooter in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

ATTO MORE FOR

movinglife®


ATTO
FLIGHT KIT

A safe journey with ATTO!




A


INSTALLING ATTO FLIGHT KIT FOR ULTIMATE PROTECTION

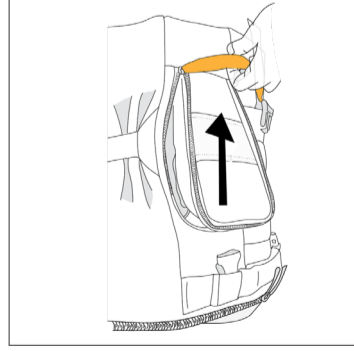


1 Pull to open zipper slider to ensure battery space filler is free (don't forget to remove your ATTO battery)

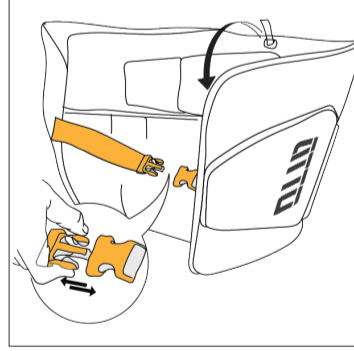


2 Unbuckle flap

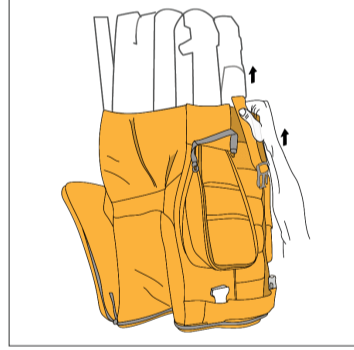


3 Unfold Flight Kit and slip over folded ATTO from the top

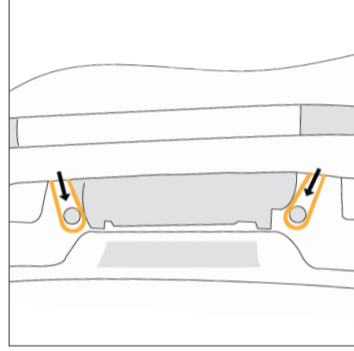


4 Connect the two loops to the hooks on the ATTO backrest to secure Flight Kit positioning

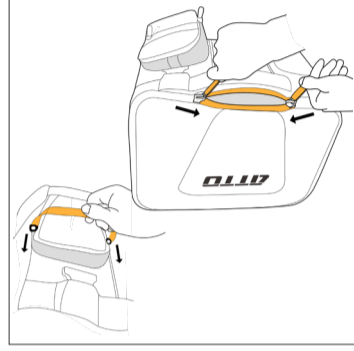


5 Close zippers on top flap and battery space filler

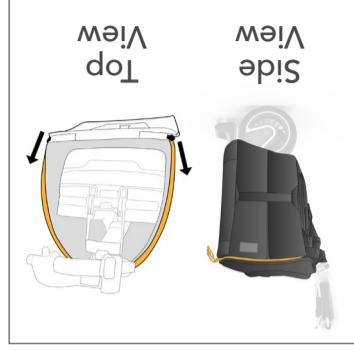


6 Or extend ATTO's trolley handle and then close the zippers around it

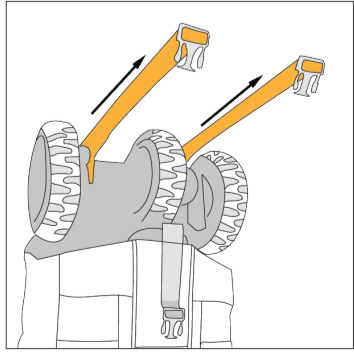


7 Convey the two straps under ATTO chassis between the wheels

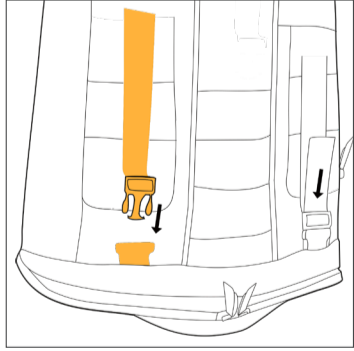


8 Thread the straps through the two dedicated loops on the pockets and buckle up

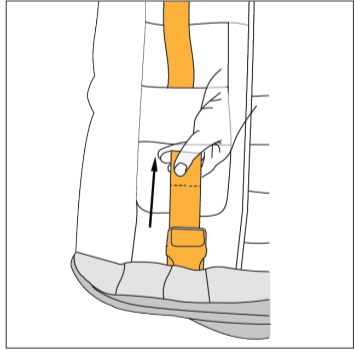


9 Pull straps to secure in place

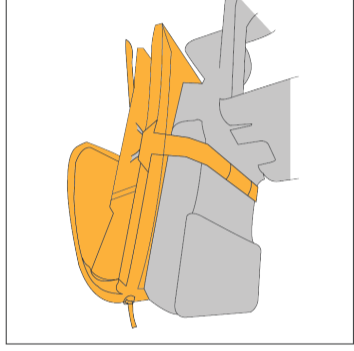

1 Fold ATTO Flight Kit, fold flap, buckle all straps out of the way and slip over ATTO backrest. Alternatively store under your seat by slipping the sleeve over it

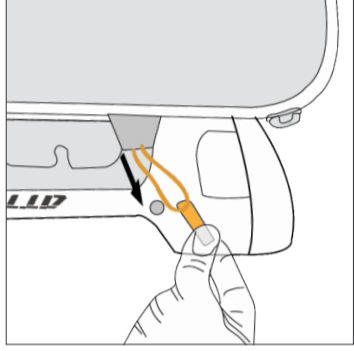

2 Connect the two loops to the hooks on the backrest to ensure Flight Kit positioning

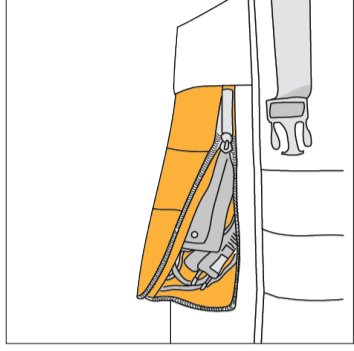

* Note additional pocket available to storage charger

CARRYING ATTO FLIGHT KIT WHILE IN DRIVING MODE

B